# Owner's manual

Balder Junior

Balder Finesse

Balder Liberty

## Fjöldi farþega

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## 7. STORAGE

### **WARNING**

**Never leave the batteries standing for a long time fully discharged. If the chair is to be stored for a period of time, the batteries must be fully charged once a month.**

Your Balder wheelchair must be kept dry, protected from the elements and preferably kept indoors. If you store your chair for any length of time without it being used, the batteries must be disconnected.

**NOTE**

There is no risk involved in leaving the charger connected to the wheelchair as long as the power is switched on. This will ensure that the batteries are charged fully and will maintain their charge.

**When the batteries are to be charged:**

1. Switch off the power to the chair.
2. Connect the charger cable to the charger contact (1)
3. Connect the power supply cable to the mains.

**When the batteries are fully charged:**

1. Disconnect the charger from the mains.
2. Disconnect the charger from the wheelchair (1).

**8.2.2. Frame and body cleaning**

The chair is cleaned using mild soapy water as required.

**8.2.3. Seat cleaning**

Cleaned as required; cf. the cleaning instructions sewn onto the seat pads.